

Helpful hints

iQ-VIEW takes too long to open a study. What can I do?

First of all, it needs to be clarified under what circumstances iQ-VIEW is considered working too slowly when opening a study. According to our definition, this is the case if iQ-VIEW takes more than 3 seconds to show the first image.

You can test the time to first image (TTFI) as follows: Select a CR/CT study from the remote archive and load it into iQ-VIEW/PRO. The first image must be visible within 3 seconds.

In case the application takes longer than 3 seconds, here is a list of possible reasons and the corresponding solutions.

Reason	Solution
Network	
Is iQ-VIEW called from HIS/RIS/EMR systems?	→ iQ-VIEW.exe is slowed down due to the license management. Use call-helper.exe instead of iQ-VIEW.exe with parameters to start iQ-VIEW ≥ 3.0. See iQ-VIEW Administration Guide sections 9.8 and 9.9.2 for details.
If iQ-VIEW is called from a HIS/RIS/EMR, does iQ-VIEW have to retrieve the study from a remote DICOM node first?	→ A Gbit network connection doubles the speed compared to a 100 Mbit connection
	→ Using enhanced DICOM for CT/MR works 3-4 times faster than single-frame DICOM
	→ A large number of locally stored studies extends the DICOM retrieval time heavily. Delete local studies faster e.g. after 3 days by activating the overflow management in the IQSERVER or move the KPSTUDY.DIR database file to an SSD to increase the speed.
Software (internal/external)	
Is the Debug logging mode active?	→ Active Debug logging can reduce the overall performance. Only use this mode temporarily for troubleshooting. Otherwise switch it off.
Is an antivirus program running on the iQ-VIEW workstation?	→ An antivirus program may slow down iQ-VIEW if it checks inbound DICOM data. Configure the antivirus program to exclude checking DICOM traffic and the iQ-VIEW DICOM data directory.
Hardware	
Slow visualization on monitor?	→ Best use nVidia or AMD graphics cards with a bus width of 256 or 384 bit (see Administration Guide section 2.2).
	→ Use manufacturer's graphics driver and not the Microsoft graphics driver.
Delayed opening of studies stored on HDD?	→ Use SSD to double the speed.

Stamp of sales partner



IMAGE INFORMATION SYSTEMS EUROPE GMBH
LANGE STR. 16 | 18055 ROSTOCK | GERMANY
TEL. GER: +49 381 496 5820 | TEL. INDIA: +91 33 4600 0657
TEL. US: +1 470 300 4480
WWW.IMAGE-SYSTEMS.BIZ | INFO@IMAGE-SYSTEMS.BIZ